

ZOLL AED Plus # DAHS-NSCZAEDP

Name:	Employee ID #:
Unit:	Title:

PERFORMANCE CRITERIA - Unless otherwise specified all skills will be demonstrated in accordance with the appropriate UC Davis Health Policy and Procedure.

These skills will be considered complete when all below performance criteria are completed and pages 1, and 2 have been scanned and emailed to: cpgn@health.ucdavis.edu

Method of Instruction	Preceptor Verification of Skill: Method of Evaluation
CP=Clinical Practice D=Demonstration E=Education Session OM=Online Module P=Policy/ Procedure Review SP=Study Packet REF=Reference Information only – not assigned in UCL	N/A=Not applicable to specific patient care area O=Observation (in clinical setting) OT=Online Test RD=Return Demonstration T=Written Test V=Verbal

ZOLL AED Plus Checklist #DAHS-NSCZAEDP	Method of Instruction	Method of Evaluation	Date	Verifier Initials
References: UC Davis Health Policy 6005: Automated External Defibrillator (AED-Zoll)				
DEMONSTRATE SET UP				
Locates Status Indicator which indicates that AED Plus is ready to be used GREEN ✓				
Locates power button and turns ON device				
Follows AED Plus prompts				
SELECTS AND CORRECTLY APPLIES APPROPRIATE DEFIBRILLATOR PADS				
Selects appropriate pads for adult or pediatric patients				
Reviews preparation of clean, dry skin surface for application of pads				
Reviews special circumstances that could affect pad placement, including water, implanted defibrillators/pacemakers, and transdermal medication patches				
Demonstrates proper pad placement for CPR-D-padz® or CPR-Stat padz® for adults/Pedi-padz® II				
Connects defibrillator pads to the AED Plus® connector (if not already pre-connected to device)				
DEMONSTRATES PROPER STEPS FOR ANALYZING THE RHYTHM				
Emphasizes verbal and visual cues and importance of not touching patient during rhythm analysis				
Demonstrates visual confirmation that no one is touching patient prior to pressing illuminated				
Demonstrates principles of safe defibrillation (does not touch pads, patient, or any conductive materials)				
Performs high-quality CPR immediately after any shock delivery or if no shock advised				
Identifies audio and visual indicators of Real CPR Help® (compression rate metronome, compression depth)				
Verbalizes duration of compression cycle before device prompts user to stand clear for rhythm analysis (2 MINUTES)				

ZOLL AED Plus # DAHS-NSCZAEDP

Name:	Employee ID #:
Unit:	Title:

PERFORMANCE CRITERIA - Unless otherwise specified all skills will be demonstrated in accordance with the appropriate UC Davis Health Policy and Procedure.

These skills will be considered complete when all below performance criteria are completed and pages 1, and 2 have been scanned and emailed to: cppn@health.ucdavis.edu

ZOLL AED Plus Checklist #DAHS-NSCZAEDP	Method of Instruction	Method of Evaluation	Date	Verifier Initials
MAINTENANCE AND CARE				
Explains that AED Plus can be cleaned with soap and water or 90% Isopropyl alcohol after use				
Ensures that electrodes are not expired				
Emphasizes the importance of visually checking Status Indicator window to ensure that device: GREEN ✓				

PRECEPTOR SIGNATURE:

Signature and Printed Name of Verifier (preceptor or other verified personnel) who have initialed on this form:

Initials:	Print Name:	Signature:

PRECEPTEE STATEMENT AND SIGNATURE:

I have read and understand the appropriate UC Davis Health Policies/Procedures and/or equipment operations manual, I have demonstrated the ability to perform the verified skills as noted, and I have the knowledge of the resources available to answer questions.

Name:	Signature:	Date: